



Parker Hannifin Corporation
 Porter Instrument
 245 Township Line Road
 Hatfield, PA 19440
 Office 215 723 4000
 Fax 215 723 5106

REF: 92120041
19mm Corrugated Scavenger
Hose Assembly Instructions

Contents:

19mm Corrugated Hose
 5 Ties (Plastic illustrated
 Or Hook and Loop provided)



1: Firmly push one end of 19mm Corrugated Hose onto the Nitronox Scavenger.



2: Connect the Scavenger Collector Manifold to the Nitrous Oxide Demand Valve.

3: Connect the 19mm Corrugated Hose to the Scavenger Collector Manifold and align parallel with Fresh Gas Delivery Hose.



4: Fasten Plastic Ties (or Hook & Loop) starting a couple of inches below the Demand Valve and then approximately every 12 inches.

5: When replacing 19mm Corrugated Scavenger Hose, carefully cut plastic ties with scissors (or remove Hook & Loop Ties).

The Quality System for Porter Instrument is certified to ISO 13485.

Check our website: www.porterinstrument.com/nitronox

To register your product: www.porterinstrument.com/resources-dental choose Warranty tab.

To download a User's Manual: www.porterinstrument.com/resources-dental choose Manuals tab